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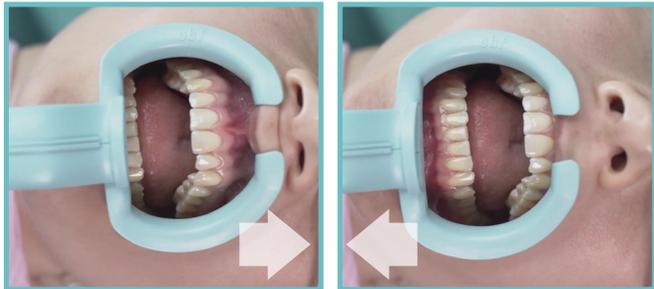
Published online 2021-09-12

Perioral Aerosol Sequestration Suction Device Effectively Reduces Biological Cross-Contamination in Dental Procedures

Victor Llovet¹, Maria Laura Gouveia², Vicente Lozano-de Luaces³, Maria Cristina Manzanares⁴

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Abstract

Objective: The reduction of during dental procedures is a common concern for dental professionals which has increased due to coronavirus (severe acute respiratory syndrome coronavirus 2) pandemic. The development of devices to specifically mitigate cross-contamination by droplet/aerosols is critical to stop infection transmission. The objective of this study is to assess the effectiveness of a perioral suction device (Distal Suction OBF) to reduce biological contamination spread during dental procedures.

Materials and Methods: Forty patients were randomized 1:1 to a standard professional dental hygiene treatment with OBF and without. Airborne respiratory (ART) bioluminescence assay was used to evaluate the spread of potential contaminants. The total number of viable light units (RLU) from key dental operating locations (operator's face shield, back of the surgical operator's gloves, patient's safety goggles, and instrumented table) were measured. Percentage contamination reduction between control and OBF were compared.

Statistical Analysis: Primary outcomes, total RLU, was analyzed by comparing the means of log₁₀ data, using a two-sided two-sample t-test. Secondary outcomes are RLU of log₁₀ data for the different locations were analyzed in the same way. Proportion of patients from whom different locations reported events (leak, accept-able, and failed) were analyzed by using Fisher's exact test.

Results: For the whole dental environment, RLU reduction (>150 units) achieved with OBF was 98.4% (95% CI: 93.9-100). In dental operators location the reduction in RLU

← Zugang zur Studie.
Quelle NCBI: National Center for Biotechnology Information